



ABADISIN[®]

ANTIBACTERIAL DISINFECTANT CLEANER

U.S. Patent No. 5,444,094 EPA REG. No. 1839-83-72072



Description:

Abadisín[®] is formulated to disinfect, clean & deodorize hard, non-porous, inanimate environmental surfaces such as: metal surfaces, stainless steel surfaces, glazed porcelain, glazed ceramic tile, plastic surfaces, chrome, brass, copper, laminated surfaces and baked enamel surfaces. Ideal for medical equipment, hospital beds, high contact areas, counters, operating tables, telephones, door knobs, sinks, shower stalls, bathtubs, cabinets & drawers.

Directions:

DISINFECTION, DEODORIZING AND CLEANING - Remove gross filth or heavy soil prior to application of the product. Hold container six to eight inches from surface to be treated. Spray area until it is covered with the solution. Allow product to penetrate and the surface to remain wet for 3 minutes (for Bloodborne Pathogens: allow surface to remain wet for HIV-1 1 minute and for HBV and HCV 5 minutes). No scrubbing is necessary. Wipe off with a clean cloth, mop or sponge. This product will not leave grit or soap scum. **TUBERCULOCIDAL ACTIVITY** – This product exhibits disinfectant efficacy against Mycobacterium tuberculosis BCG (Mycobacterium bovis) at 20 degrees Centigrade (20°C) when the treated surface is allowed to remain wet for 5 minutes.

MILDEWSTAT - To control mold and mildew on pre-cleaned, hard, non-porous surfaces spray surface to be treated making sure to wet completely. Let air dry. Repeat application at weekly intervals or when mildew growth appears.

Specifications:

Appearance.....	Clear liquid
Odor.....	Pleasant odor
Specific Gravity.....	1.010 ± 0.005
pH.....	12.0 ± 0.5
Stability.....	One year minimum
Boiling Point.....	Appx. 212°F
Flash Point.....	212°F

*Efficacy Data available at www.britestarp.com

Distributed by:



Mayagüez
787-833-8440

Hato Rey
787-767-2562

Product Code: BS310-QT

Packaging: 12 Bottles (32oz.) / Case

Used For:

Medical Equipment

Patient Beds

High Contact Areas

Bathroom

